

Meditelle Cleaning Instructions



Couches, Chairs & Trolleys

Framework

- Steel Powder Coated

Regular cleaning with warm soapy water and a soft cloth. A mild disinfectant may be used in dilution. Do NOT use undiluted strong detergents, abrasives or solvent / chemical cleaners on powder coated surfaces. To avoid delamination and rust creepage, you must avoid using any chlorine releasing agent in excess of 1-10,000 ppm. Never apply full strength bleach. Always rinse well with clean water and dry surfaces thoroughly after cleaning. NHS approved cleaning wipes such as Clinell and Vernacare Tuffie 5 Sanitising Wipes are acceptable. Do NOT use any form of hydrogen peroxide or ammonia – even dilute, as this will cause movement of the coating and subsequent rust.

Tips:

- Care should be taken when cleaning floors in areas around framework with a strong chlorine or bleach agent in excess of 1-10,000ppm as this will react with the coated and steel surfaces and cause them to perish.
- We recommend to periodically oil axles and any moving parts with a liquid grease spray e.g. WD40

- Aluminium

Aluminium should be cleaned using a soft damp non-abrasive cloth and polished completely dry. Exposure to wet or humid conditions could result in discolouration or corrosion. Keep all surfaces dry at all times. Do NOT use Bleach, synthetic detergents, solvents, wax polishes or aerosol sprays.

- Stainless Steel

Stainless steel should be cleaned using a soft damp cloth and polished completely dry. Do NOT use an abrasive cloth. Constant use of certain cleaning products such as Iodine may result in permanent damage to the metal surface and should not be used. Exposure to wet or humid conditions could result in discolouration or rust marks even on 'stainless' steel. Keep surfaces dry at all times. Do NOT use Bleach, synthetic detergents, solvents, wax polishes or aerosol sprays.

Chair & Stool Bases

Keep base and wheels free from dust and grease by wiping with a dry cloth. Clean with warm soapy water and a soft cloth or anti-bacterial wipes such as Clinell. Mild disinfectant may be used in dilution. Do NOT use undiluted strong detergents, abrasives, solvent / chemical cleaners or bleach.

Electrical Components

Keep the unit free from any dust and grease by regularly vacuuming and wiping with a clean dry cloth. Electrical components should NOT be exposed to any form of liquid or humid conditions. Plastic casings of actuators or handsets may be cleaned using a soft damp non-abrasive cloth and polished dry. Bleach, synthetic detergents, solvents, wax polishes or aerosol sprays should NOT be used.

Note: For safety electrical equipment must be disconnected from the mains before cleaning.

NOTE: CONSTANT WATER EXPOSURE IN DAMP / HUMID & WET AREAS CAN AFFECT THE INTEGRITY AND LIFE SPAN OF FRAMEWORK AND ELECTRICAL COMPONENTS. IF USING IN / NEAR A BATHING AREA ENSURE THE PRODUCT IS THOROUGHLY DRIED BETWEEN USE.